


Symbol-Based

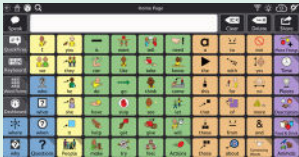

TD Snap® Motor Plan INCLUDED

Core approach designed using motor planning principles, organization of fringe words by form and function, and intuitive PCS symbols. Focused on:


- Generative language concepts (brick-by-brick sentence building)
- Some common phrase-based communication (QuickFires)
- MP 30 optimized for eye gaze



TD Snap® Core First INCLUDED

Core and pre-stored messages (Topics and QuickFires) designed with fringe words organized by theme or event.


- Two ways to grow (location centered or by grid size)
- Topics organized by communicative intents: Questions, General Comments, Positives, Negatives

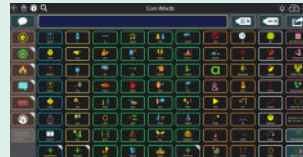

TD Snap® Core First High Contrast INCLUDED

INCLUDED


Designed for individuals with low vision using Core First organization. The majority of symbols are high contrast, backgrounds and buttons are dark, and text is reduced or removed to decrease visual complexity.

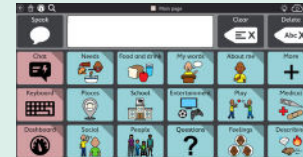

TD Snap® Express INCLUDED


Designed with a focus on simplicity and ease of use while supporting movement forward in communication and language skills. Each Theme or context contains common sentence starters, single words, and full messages.

Aphasia

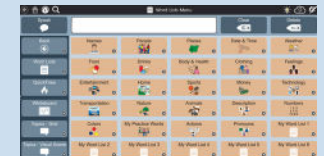

TD Snap® Aphasia INCLUDED


Designed specifically based on the needs and strengths of people with aphasia by focusing on providing visual context and focusing on word and message retrieval. Also provided: Keyboard, Visual Scenes, Whiteboard, Photo Albums, Topics, Word Lists and Rating Scales.

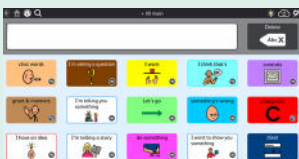

TD Snap® PODD

In-app Subscription

Pragmatic Organization Dynamic Display. Designed for individuals with complex communication challenges or complex bodies that need immediate access to communicate for a variety of functions.

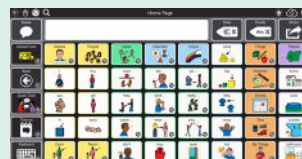

TD Snap® Gateway

In-app Subscription


Core Vocabulary designed with sentence development links/predictive navigation and sentence starters with themed pages for leisure, educational, and recurring conversational needs.



TD Snap® Core First Scanning INCLUDED


Designed for scanning using Core First organization. Vocabulary is arranged to optimize group scanning. Includes "Learn to Scan" tool available in the dashboard.

Text-Based

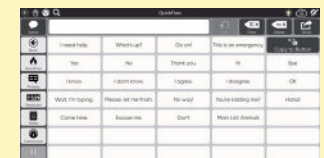


TD Snap® Text INCLUDED

Designed for literate individuals. Communication tools include Keyboard, QuickFires, Phrases and Notes.



Symbol-Based

FEATURES	TD Snap® Motor Plan	TD Snap® Core First and Core First High Contrast	TD Snap® Core First Scanning	TD Snap® Express	TD Snap® PODD	TD Snap® Gateway
Creator	Tobii Dynavox	Tobii Dynavox	Tobii Dynavox	Tobii Dynavox	Gayle Porter	Joan Bruno
Languages	English (US/UK/AU), German, Spanish (ES/MX) Bilingual Support: English/Spanish (US)	Arabic, Chinese (simplified), Czech, Danish, Dutch, English (US/UK), Finnish, French (FR/CA), German (PCS/Metacom 9), Hebrew, Icelandic, Italian, Japanese (Hiragana/Kanji keyboard), Norwegian (Bokmål), Polish, Portuguese (BR/PT), Spanish (ES/MX), Swedish, Swiss German (PCS/Metacom 9), Turkish High Contrast only available in English (US). Bilingual Support: English/Spanish (US), English/French (CA)	English (US), German	English (US/UK/AU), Arabic, Chinese (simplified), Danish, Dutch, French (FR/CA), German, Italian, Japanese (Hiragana/Kanji keyboard), Polish, Portuguese (BR), Spanish (LATAM/ES), Swedish Bilingual Support: English(US/Spanish (LA)	English (US/AU), Danish	English (US)
Grid Sizes	2-3 grids: 30, 40, 66 Grid sizes are not customizable.	12 optimized grids: 1x2, 1x2, 2x2, 2x3, 3x3, 3x4, 4x4, 5x5, 6x6, 7x7, 7x9, 8x10 All grid sizes are customizable.	3 grids: 3x3, 4x3, 6x5 All grid sizes are customizable.	Single grid size: 18 buttons	PODD PODD 15 Plus Preschool PODD 15 Plus School PODD 15 School PODD 15 Preschool PODD 60 Complex PODD 60 EKW	Child Functional (3x3, 4x5, 5x6, 6x10, 8x10, 9x11), Teen/Adult Functional (3x3, 4x5, 5x6, 5x8, 6x10, 8x10, 9x11), Child Language and Literacy (3x3, 4x5, 5x6, 5x8, 6x10, 8x10, 9x11), Advanced Communicator (5x8, 6x10, 8x10, 9x11), Text (5x8, 6x10, 8x10, 9x11), Gateway Pro 11x13
Word Count	2,700 words not including inflectional forms (more verbs than Core First). 7600 with word forms. Same regardless of grid size.	2,700 words not including inflectional forms (more nouns than Motor Plan). 6,570 with word forms. Grows with grid size.	2,700 words not including inflectional forms. 6,570 with word forms. Grows with grid size.	1300 words and phrases.	1600-3300 words depending on grid size selected. Grows with grid size.	800 to over 2000 words depending on grid size. Grows with grid size.



Symbol-Based

FEATURES	TD Snap® Motor Plan	TD Snap® Core First and Core First High Contrast	TD Snap® Core First Scanning	TD Snap® Express	TD Snap® PODD	TD Snap® Gateway
Whole Message Organization	QuickFires, No Topics (can be added)	QuickFires, Topics-by context or activity	QuickFires, Topics-by context or activity	Message builders, whole messages in each Theme, Chat	Chat Words, Pragmatic function organization	Quick Chat, Themes by context or activity
Toolbar	On home page only	On all pages	On all pages	On all pages	No	On all pages
Vocabulary Manager	Yes: Vocabulary Filter	No: Hide words individually	No: Hide words individually	No: Hide words individually	No: Hide words individually	No: Hide words individually
Path to get to words	Only 1 path (2 exceptions: orange, may)	More than 1 path	More than 1 path	More than 1 path	More than 1 path	More than 1 path
Grammar support (Auto morphology/Manual morphology)	Yes: Auto morphology and Manual morphology. TD Motor Plan (English) includes an advanced automatic morphology system that uses artificial intelligence to make better predictions. <ul style="list-style-type: none"> ■ Looks at preceding 12 words ■ Treats all verbs individually 	Yes: Auto morphology and Manual morphology. TD Core First automatically inflects words to an appropriate form, based on preceding context. <ul style="list-style-type: none"> ■ Looks at preceding 3 words ■ Treats all verbs the same 	Yes: Auto morphology and Manual morphology. TD Core First automatically inflects words to an appropriate form, based on preceding context. <ul style="list-style-type: none"> ■ Looks at preceding 3 words ■ Treats all verbs the same 	None	Only in PODD 60: Word forms provided on a separate page	Yes
Keyboards	Automatically grid matches with QWERTY, ABC options	QWERTY, ABC, AEIOU, Grid Matching, Frequency	AEIOU, QWERTY, Frequency	Grid Matching, QWERTY, ABC	PODD Complex and PODD EKW-QWERTY	QWERTY, ABC, AEIOU, Frequency
Supports (e.g. First Then, Schedule)	Yes (Dashboard)	Yes (Topics)	Yes (Topics)	Yes (Dashboard)	No (can import)	No (can import)
Boardmaker Player	Yes (Dashboard)	Yes (Dashboard and Toolbar)	Yes (Dashboard and Toolbar)	No	Yes (in edit and controls)	No (can import)
Whiteboard	Yes (Dashboard)	Yes (Dashboard)	No	Yes (Dashboard)	Yes (in edit and controls)	No (can import)
Google Assistant, Alexa, Siri	Yes (Dashboard)	Yes (Dashboard)	Yes (Dashboard)	No	Yes (in edit and controls)	No (can import)

Note: Editing or Customizing a button/symbol is the same across all page sets.

FEATURES	TD Snap® Text	TD Snap® Aphasia
Creator	Tobii Dynavox	Tobii Dynavox
Languages	English (US/UK/AU), Spanish	English (US), German, Spanish (LATAM)
Grid Sizes	1 pre-made grid size: 7x10 Grid size is customizable .	3 pre-made grids: 3x3, 4x3, 6x5 All grid sizes are customizable .
Toolbar	On all pages	On all pages
Vocabulary Manager	No: Hide buttons individually	No: Hide buttons individually
Whole Message Organization	Phrases-by context or activity	Topics-by context or activity; Visual Scene or Grid Based
Quick messages for use in any context	QuickFires	QuickFires
Only 1 path to get to words	NA	NA
Grammar support (Auto morphology/Manual morphology)	NA	NA
Keyboards	QWERTY, ABC	QWERTY, ABC
Supports	No (can import)	No (can import)
Boardmaker Player	No (can import)	No (can import)
Whiteboard	Yes (Dashboard)	Yes (Toolbar)
Google Assistant, Siri, Alexa	Yes (Dashboard)	Yes (Dashboard)

Note: Editing or Customizing a button/symbol is the same across all page sets.